



Operating Manual for Audio Patient Monitor

Startup

Plug in sensor in the right socket "SENSOR" on the flipside of the device.
Connect the device with the power supply or the integrated battery.
Connect the Esophagus-Stethoscope with sensor and adjust it to your usual volume "VOLUME".
Please be sure to switch off the device after using: The volume controller should be switched to OFF.

Charging Battery

To charge the battery, simply plug the power supply into the right socket "POWER". Please be sure that the power supply is adjusted to 9V DC; also, polarity should also be correctly adjusted with the front contact showing the positive pole.

Connect the power supply with a socket.

The battery should now be able to charge.

The red LED "CHARGE" should be lit while charging.

Note that the power supply can be connected with the device all the times as overcharging of the battery is not possible.

A complete charge from empty to full will take approximately 10 hours.

If the red LED does not light up, check the voltage and polarity from your power supply.

Please Note

You should only put the device to storage when the battery is full or close to full.

Switch off the device after every use.

It is advisable to connect the device to the power supply when not in use (charging the battery).