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1) Packing list

The following parts should be included in a complete EICKEMEYER ultrasonic scaler:

Description	Dimensions	Quantity
Main unit	205mm 135mm 74mm	1
Handpiece		1
Power supply	120mm 75mm 65mm	1
Foot pedal	100mm 62mm 36mm	1
Scaling tip	#01	2
	#02	2
	#03	1
Water line	6mm	1
Wrench	27mm 12mm	1
Three-way connector	2.5mm 2.5mm 2.5mm	1
Packing list		1
Instruction manual		1

2) Scope of application

For veterinary use only.


EICKEMEYER ultrasonic scaler is used for dental calculus elimination.

3) Main technical specifications

Power supply:	~220V - 230V, 50Hz, 150mA, Class II equipment, B-Type	
Main unit input:	~24V, 50Hz, 1.3A	
Output power:	3~20 W	
Vibration frequency:	30 ± 3 KHz	
Fuse rating:	F1: 250V T0.5A	F2:250V T1.6A
Water pressure:	0.01~0.5 MPa	
Main unit weight:	0.85 Kg	Power supply weight1.25 Kg
Intermittent operation:	6 min / 8 min	
Do not use in the presence of flammable anaesthetic gas, oxygen or nitrous oxide.		
Working Condition	Environment temperature:	5° - 40°
	Relative humidity:	80%.

4) Components

The components of the equipment are as showed below:

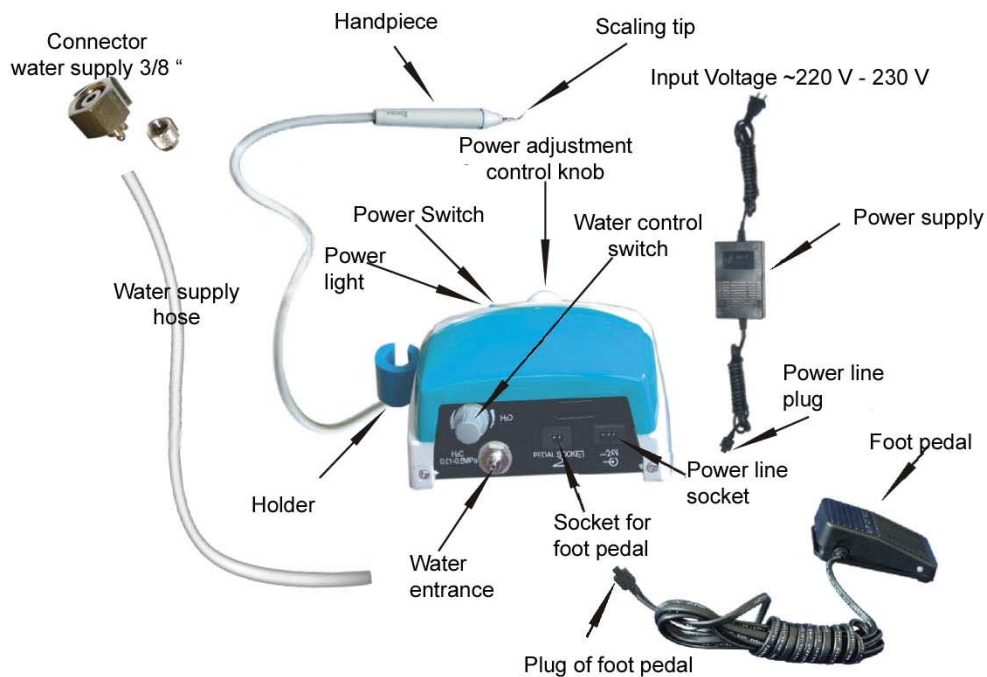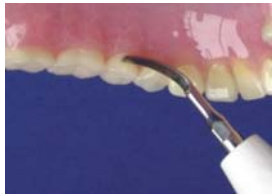
5) Installation

- 1) The use of the Eickemeyer Dental Scaler is restricted to qualified personal.
- 2) Open the packing box, make sure that all the parts and accessories are complete according to the packing list.
- 3) Take the main unit out of the box and put it on a plane and stable surface.
- 4) Turn the water control switch counter-clockwise towards its max imam position and turn the control knob towards a position 1 power adjustment.
- 5) Connect the foot pedal into its socket.
- 6) Connect one end of the water line to the water entrance and the other end to the water source.
- 7) Connect the power supply.
- 8) Press the power switch of the main unit, then the power indicator shines. LED will be on.

Warning: It is necessary to turn off the water supply before leaving the unit. The water supply hose needs to be checked for leakage on a regular basis.

6) Operation method

- 1) Select a tip, attach it to the handpiece and tighten it with the wrench. In order to obtain good vibration of the tip it must be tightened properly. The tip should be removed after each use to avoid becoming locked.
- 2) Set the power adjustment control to 1 and turn the unit on by pressing the power switch.
- 3) Adjust water volume: Step on the foot pedal, the tip begins to vibrate, then turn the water control switch to form a fine spray to cool down the handpiece.
- 4) Vibrating intensity: Adjust the vibrating intensity by turning the power adjustment to a middle grade. Due to different rigidify of the tartar, the vibration needs to be adjustment with each patient differently.
- 5) After scaling, keep the machine working for about 30 seconds with the water supply on, so that the handpiece and the scaling tips can be cleaned.

<p>01#</p> 	<p>01#</p>  <p>To eliminate the supragingival tartar, with the edge of the tip touch the surface of the teeth</p>	<p>01#</p> 
<p>02#</p> 	<p>01#</p>  <p>To eliminate the supragingival tartar, make the edge of the tip touch the surface of the teeth with linear motion.</p>	<p>01#</p>  <p>To eliminate the tartar on the dental cusp, make the edge of the tip touch the cusp with linear motion.</p>
<p>02#</p> 	<p>02#</p>  <p>To eliminate the big supragingival tartar, use the incisal edge of the tip to touch the surface of teeth.</p>	<p>02#</p> 
<p>03#</p> 	<p>03#</p>  <p>To eliminate the subgingival tartar, make the edge of the tip to touch the surface of teeth with linear motion</p>	

7) Tips

- 1.) Tip #1 can be used in most cases in small animals. It permits easy access to all tooth surfaces. The tip should be used tangentially to the tooth at an angle less than 20°. Please note that strong pressure will damage the tip. It should be slightly run across the surface.
Never use the tip perpendicular to the tooth surface.
- 2.) Tip #2 can be used to remove big resistant tartar.
- 3.) Tip #3 is a finer tip and can be used in most cases. It may be especially used for supragingival and sub-gingival scaling in pockets.

8) Sterilization

- 1) The scaling tip and wrench can be cleaned with ultrasonic cleaner.
- 2) All the scaling tips can be disinfected by any standard disinfectant. It may also be autoclaved or sterilized with hot air.
- 3) Handpiece can be disinfected by any mild disinfectant.
- 4) The handpiece can not be autoclaved or sterilized with hot air.



9) Warnings

- 1) **Keep the scaler clean before and after operation.**
- 2) **The scaling tip, wrench and handpiece must be sterilized before each treatment.**
- 3) **Don't screw or unscrew the scaling tip while stepping on the foot pedal.**
- 4) **The scaling tip must be fastened. There should be fine spray coming out from the tip when operating.**
- 5) **Replace damage pr worn-out tips immediately.**
- 6) **Don't make the tip twist or rub it.,**
- 7) **Don't use unclean water source, normal saline can't be used as clean water source.**
- 8) **If use water source without hydraulic pressure, the water surface should be one meter higher than the head of the patient.**
- 9) **After operating, turn off electrical source, then pull out the plug.**
- 10) **The use of other tips supplied by Eickemeyer might damage the unit void warranty.**
- 11) **The patient or doctor who uses artificial pacemaker is forbidden to use this equipment.**
- 12) **The heart disease patient, pregnant woman and children should be cautious to use the equipment.**

10) Storage and Maintenance

The equipment should be handled carefully and stored in a cool, dry and ventilated place.

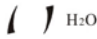11) Symbol instructions

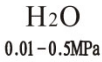	Class II	IPX1	Drip-proof
	Type B equipment		CE marked product
	Alternating Current		Foot pedal interface
	Notice! Please read instruction		Water control switch
	Used indoor only		~24V power supply < input interface
IPX0	Ordinary equipment		Water entrance pressure 0.01 - 0.05 MPa